

# **User manual for VedaQuantum 2.0**

LLC BIOKVANT  
2023

## Contents

1. Intended use
2. Direction for use
3. Device functionality
4. User guide
5. Safety measures
6. Contraindications for cupping mode
7. Terms of warranty service
8. Technical parameters
9. Troubleshooting
10. Warranty certificate for VedaQuantum 2.0

### 1. Intended use

VedaQuantum 2.0 is a wireless multifunctional physiotherapy device with multiparametric correction of the functional state through reflexogenic zones. It combines the effects of four factors: mechanotherapy (cupping), temperature (heating), magnet therapy and quantum therapy. It can be used in combination with the software modules Quantum Physiotherapy, Cupping Therapy, Osteopathy of the VedaPulse Professional software as part of the creation of a personalized recipe for stimulating reflexogenic zones and acupuncture points.

### 2. Direction for use

**Quantum physiotherapy** consists in using the combined effects of electromagnetic fields, quantum and wave effects (red light) and temperature irradiation for therapeutic and preventive purposes.

The method is effective for the prevention of exacerbation of chronic inflammatory and/or dystrophic processes of various localizations, as well as for exposure in the acute and subacute period of the inflammatory process to achieve faster remission.

It is used in conjunction with the Quantum Physiotherapy software module.

**Cupping** is based on the mechanical compression of an area of skin by a vacuum factor, creating a zone of reflexogenic hyperemia that stimulates blood and lymph circulation, interstitial drainage and nerve endings. In this mode, Cupping function is used; it is possible to add other functions: Chromotherapy, Heating, Magnet therapy.

It is used in combination with the Cupping Therapy module.

**Gua Sha massage** reduces oedema, helps increase blood and lymph circulation, improves metabolism in skin cells and improves complexion through mechanical action and activation of metabolism. Gua Sha relieves tension and tones the skin. In this mode, Cupping and Magnet therapy functions are used, it is possible to supplement the other functions: Chromotherapy, Heating.

It is used in accordance with the techniques of Gua Sha massage.

**Lymphatic drainage massage** has a curative effect on the body: it eliminates toxins, improves blood circulation, strengthens the immune system, helps to remove extra centimeters from the waist, buttocks and thighs, and is effective against cellulite. In this mode, Cupping and Magnet therapy functions are used, it is possible to supplement the other functions: Chromotherapy, Heating.

It is used in accordance with the techniques of lymphatic drainage massage.

**Therapeutic massage** – deep vacuum action helps to relieve tension in the cervical, lumbar and thoracic spine. In this mode, Cupping and Magnet therapy functions are used, it is possible to supplement the other functions: Chromotherapy, Heating.

It is used in accordance with osteopathy techniques.

### 3. Device functionality



### 4. User guide

1. Before carrying out the procedure, it is necessary to install a filter. A rubber fitting for the filter is placed in the recess on the underside of the device. If you are using Gua Sha, lymphatic drainage or therapeutic massage techniques, apply a cream or massage oil to the area to be exposed.

2. Press and hold the power button for 2 seconds to switch on. The display shows the battery level and the Chromotherapy mode.

### 3.1 Using the device for **Quantum Physiotherapy** and **Cupping Therapy**

Select according to the recommendations of the VedaPulse Professional software modules:

- Vacuum power using the Cupping button, 5 intensity levels are available.
- Heating level using the Heating button, 3 temperature modes are available 40-45-52°C.
- Chromotherapy on/off.

Install the device in the recommended zones for the required period of time in a consistent manner.

### 3.2 When using the device for **Gua Sha, lymphatic drainage, therapeutic massage**

- Set the vacuum power using the Cupping button, 5 intensity levels are available.
- If necessary, set the heating level using the Heating button, 3 temperature modes are available 40-45-52°C.
- If required, switch Chromotherapy on/off.

Use in accordance with the appropriate massage technique.

4. The device has a timer for the procedure – 10 minutes. After this time, the unit will emit 10 beeps and then switch off.

5. To switch off the device, press and hold the Power button for 2 seconds. Remove the filter and rinse thoroughly under running water. Do not allow water to come into direct contact with the device. Cosmetic residue on the working surface of the device can be removed with a wet cloth and wiped dry.

## 5. Safety measures

- Gua Sha, lymphatic drainage and therapeutic massage cannot be performed on dry skin! Apply a massage cream or oil to the skin.
- Do not use on the inside of the thighs, the back of the knees or the groin. Women should also avoid the breast area.
- Remove make-up from the skin before the procedure.
- The first procedures should be carried out at the 1st level of intensity.
- The procedure should not exceed 20 minutes.

## 6. Contraindications for cupping mode

- Cardiovascular diseases with severe cardiovascular insufficiency
- Oncology
- Blood clotting disorders
- Renal and hepatic insufficiency
- Varicose veins, thrombophlebitis
- Skin disorders (dermatitis, eczema, etc.)
- Altered skin integrity (wounds, fresh surgical sutures)
- Large pigmented lesions and moles in the area of intended exposure
- Presence of a pacemaker
- Pregnancy

## 7. Terms of warranty service

1. Do not use the device until it is fully charged.
2. Do not expose to direct sunlight.
3. Do not store near heat sources or in places subject to extreme heat.
4. Keep away from moisture.
5. Do not drop.
6. Keep out of the reach of children.

## 8. Technical parameters

Rated power, W	8
Accumulator capacity, mAh	1200
Charging voltage, V	5
Body material	ABS-plastic
Operating time, h	2
Weight, gr	300
Maximum pressure created, kPa	65

## 9. Troubleshooting

Follow the troubleshooting instructions below. Never disassemble the unit yourself to maintain your warranty. If you cannot solve the problem by following the

instructions below, contact your service representative: [support@vedapulse.com](mailto:support@vedapulse.com)

<b>Problem</b>	<b>Probable cause</b>	<b>Troubleshooting</b>
Failure to turn on the device	Battery-low	Charge the device
	Device is under charging	Use the device after full-charged
Failure to charge the device	Device incorrectly connected to charging line or plug	Reconnect the charging line and plug

## 10. Warranty certificate for VedaQuantum 2.0

Buyer .....

Phone number.....

Address.....

.....

The certificate confirms the readiness of the device for operation and grants the owner with the rights for warranty maintenance during the term specified in the passport.

Warranty obligations. Warranty period is 12 months from the date of sale of the device to the buyer. Provided that the requirements of storage, use and transportation guidelines during this term are followed, the manufacturer will repair the device at its own expense. More details can be found on the website [www.vedapulse.com](http://www.vedapulse.com)

I have no complaints to the appearance, completeness and operation of the device. I hereby confirm that I have read, understood and agree with all of the above warranty conditions.

Manufacturer

Date.....

Manufacturer Signature.....

Seller

Date.....

Seller Signature.....

Buyer

Date.....

Buyer Signature.....

Reason of device acceptance for repair

.....

.....

Information on the completed repair

.....

.....

Biokvant LLC.

630108, Novosibirsk, 1st pereulok Parkhomenko 14

tel: +7-383-334-01-79